


Mounting Instructions

Step 1:

Back the mounting lug adjustment screws all the way out to remove the tube mount assembly from the collimator. Install the tube mount assembly on the x-ray tube with the stop-pin cutout to the front as shown in diagram at right.

Use the tube mount shims provided in the manual to install the tube mount assembly to a focal spot distance of 2.44 inches per the x-ray tube manufacturers specifications.

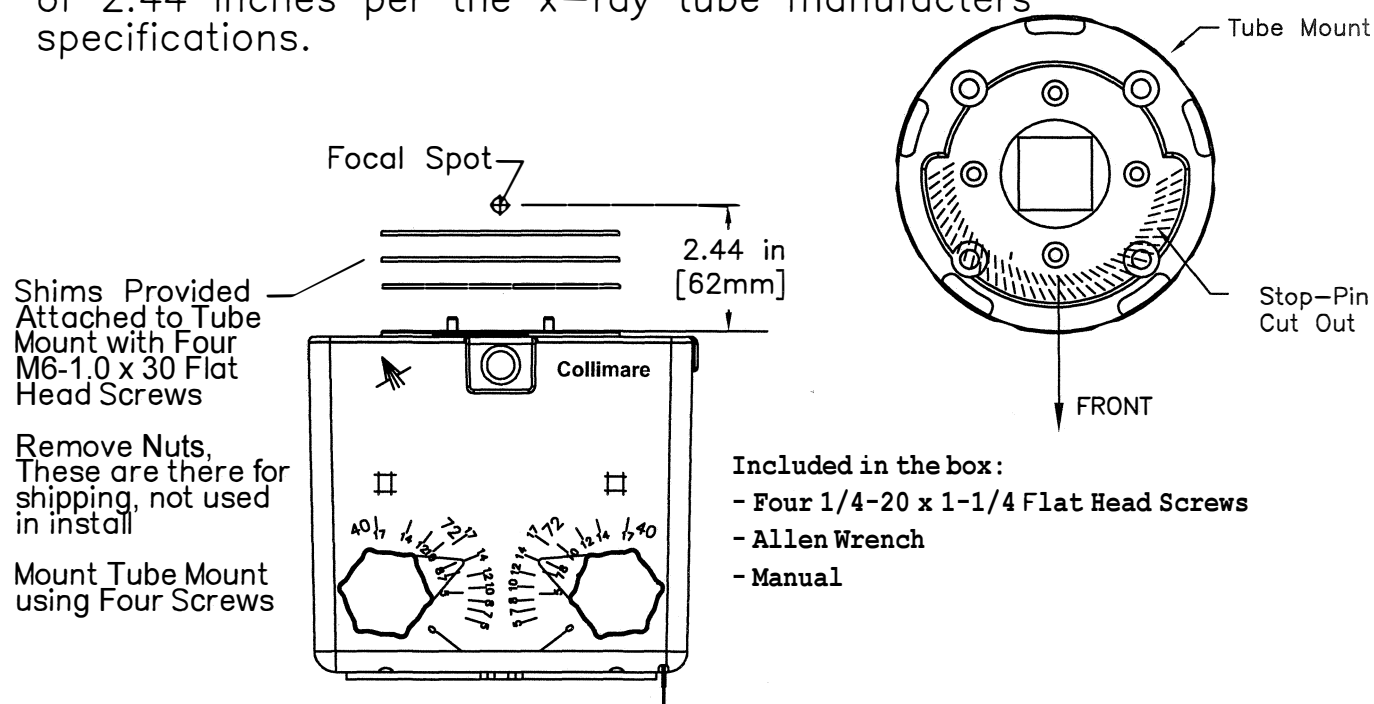

Step 2: Use an 1/8in Allen wrench

Lift the collimator up to engage the mounting ring and tighten the mounting lug screws to an even pressure. Once the screws are tight, back off each screw 1/8 turn and rotate the collimator to align the laser cross hairs with the table or bucky.

Step 3:

To align the light field and x-ray field refer to pg.2 of this fold out and follow it's instructions.

